

PETG FACE SHIELDS



DESCRIPTION:

These face shields are perfect for medical professionals or anyone needing to provide an impermeable barrier against liquids. One size fits all, fully adjustable with a rubber head strap. Not bulky so it easily fits over existing PPE garments.

Features:

Impermeable Barrier Against Liquids
Adjustable Head Strap
Can Be Sanitized for Re-Use

Size:

Approx 12" W x 11" H

Materials:


Shield - 0.04" PETG
Fixed Straps - 0.06" PETG
Adjustable Strap - Latex / Clay Mixture

Price: \$10ea

Case Quantity: 40

Availability: 1 Week Lead Time

Ships From: Naples, Florida

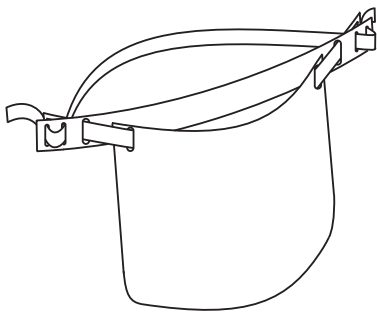

Insert the ends of the rubber strap through the second slot in the short strap. Pull the shield over your head and pull the ends of the rubber strap for a snug fit.

INSTRUCTIONS:

- The user is solely responsible for the safe and appropriate use of this Device.
- This Device is shipped without sanitization. It must be cleaned and sanitized with soap, water, and a broad band disinfectant prior to first use.
- Place the Device securely over the face so that it does not slip or move.
- This Device must be used together with such other protective appliances and clothing as needed to prevent contamination.
- The Device does not work if not worn or if not worn properly. To be of any effect, it must be worn at all times when in contact with potentially infected persons or objects.
- The Device must be cleaned and sanitized frequently, not less than after each use and after each known or suspected contact with a potentially infected person or object.
- Protective gloves must be worn when handling or cleaning this Device.
- The Device must be disassembled and safely discarded or destroyed if it is broken, unusable or cannot be cleaned and sanitized.
- This Device is for competent adult use only, and not for use by children or persons who are incapacitated in any manner.
- By using this Device, the user agrees and consents to the terms and conditions set forth in this instrument. If the user does not agree and consent to such terms and conditions, use of this Device is prohibited and it must be discarded.

DISCLAIMER AND AS-IS NON-WARRANTY:

This Device is a face shield only. It does not filter air. This Device may not prevent contact with airborne particles and/or liquids that go around the shield. It does not shield any object or part of the body that is not behind the shield.

- This is not a medical device, and it is not regulated or approved by any governmental or non-governmental agency or authority.
- Touching this device or bringing it into contact with any object can cause it to be contaminated.
- Distributor and Manufacturer do NOT guarantee this item to be virus free or contamination free.
- This device is provided As Is, with no warranty of any kind or nature.
- Use this Face Mask at your own risk

